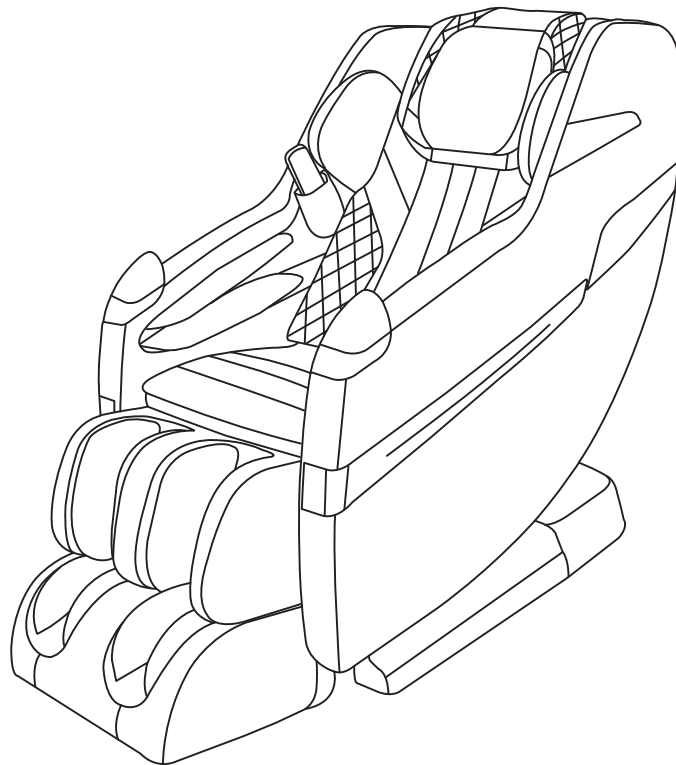
SMARTJAZZ

Energized with melody touch

Rev.1.0

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OGAWA

SMART JAZZ

Energized with melody touch

User Manual

All information is correct at the time of print

CONTENTS

| | |
|----------------------------|----|
| Safety Instructions | 03 |
| Warranty Claims | 05 |
| Product Parts | 06 |
| Moving the Chair | 07 |
| Controller Diagram | 08 |
| Product Operation Guide | 09 |
| Product Features | 14 |
| Cleaning And Maintenance | 15 |
| Questions And Answers(Q&A) | 16 |
| Troubleshooting | 17 |
| Product Specification | 19 |
| Contact Us | 20 |

Safety Instructions

Warning!

The plug must be fully inserted into an appropriate outlet. Improper connection of the equipment-grounding conductor can result in the risk of electric shock.

Check with a qualified electrician and service personnel if you are in doubt as to whether the product is properly grounded. Do not modify the product's accompanying plug. If the outlet is not suitable, be sure to have a proper outlet installed by a qualified electrician.

When using the massage chair, basic precautions should always be followed to reduce the risk of electric shock, burns, fire, or injury.

Read the following details before use:

1. Be sure to insert the massage chair plug into the proper electrical outlet.
2. Make sure that the power plug is securely inserted into the proper electrical outlet to reduce the risk of short-circuit and fire.
3. Always unplug the massage chair from the electrical outlet immediately after use and before cleaning.
4. DO NOT use the massage chair in places where the humidity is high.
5. DO NOT use the massage chair outdoors and avoid places with direct sunlight.
6. If the power goes off during use, be sure to turn the power switch to the OFF position and pull the power plug out from the socket to prevent any damages when the power comes back.
7. Never operate the massage chair if the power cord is wet or damaged.
8. Never plug/unplug the power cord from the socket with wet hands.
9. Close and continuous supervision is necessary when this appliance is used by or near children, invalids or disabled persons.
10. Use this appliance only as described in this user manual.
11. Do not use any attachments other than those recommended by OGAWA.
12. Do not stand, sit or jump in the calves rest when the massage chair is in use. This may result in an accident or injury.
13. Do not attempt to stand on the backrest and armrest .This may result in an accident or injury.
14. Be careful not to fall asleep during the massage.

Safety Instructions

15. Always keep the massage chair clean and never insert any object into the gaps on and around the massage chair.
16. Do not remove the massage chair by pulling the power supply cord.
17. Be sure to check that there is no one (children in particular), pets or obstacles under the massage chair prior to lowering the leg rest /backrest.
18. Always check whether the surface of the massage chair is torn or ripped before use. If you find any unfamiliar tears in any area of the cloth in the backrest, do not use the massage chair. Kindly contact OGAWA service center immediately for repairs.
19. This appliance is designed for household use only.
20. For those suffering from illnesses, please consult your doctor on whether you can use the massage chair if in doubt.
21. The following persons are requested to consult a doctor prior to use in order to reduce the risk of the user feeling sick or suffering from an injury:
 - Those who have a malignant tumor;
 - Those who suffer from heart disease;
 - Those who are experiencing pregnancy, menstruation and puerperal;
 - Those who require complete rest;
 - Those who are under medical treatment;
 - Those who suffer from back problems or whose backbone is abnormal or bent.
22. If you experience pain during use, stop the massage chair immediately. If any injuries happen, consult a doctor immediately. Never massage on swollen or inflamed parts of the body.
23. This product is a non-professional appliance designed to provide a comfortable massage. Do not use this product as a substitute for medical treatment.
24. Do not use this massage chair for more than 30 minutes at one time and kindly limit the massage to a concentrated area for not more than 15 minutes. Over-massaging can result in overheating, and thus reducing the product's lifespan.
25. Do not operate where aerosol (spray) products are used or where oxygen is being administered.
26. Children shall not play with the appliance.
Cleaning and user maintenance shall not be made by children without supervision.

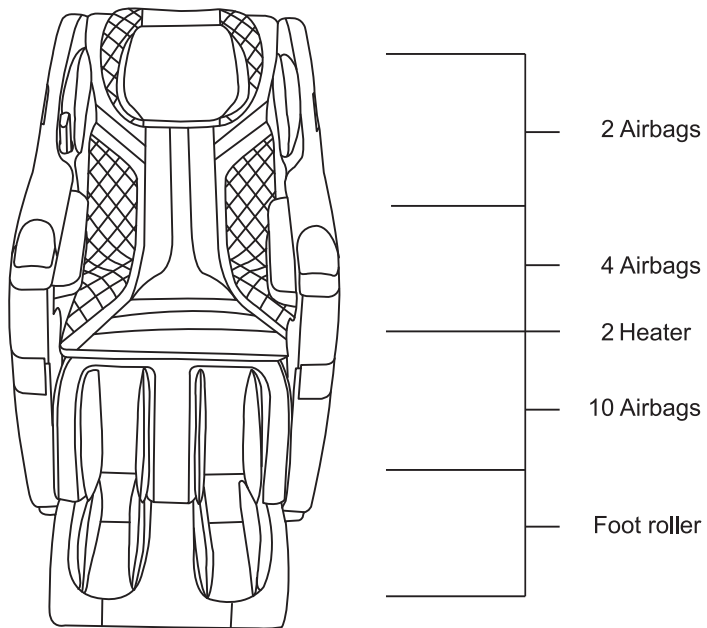
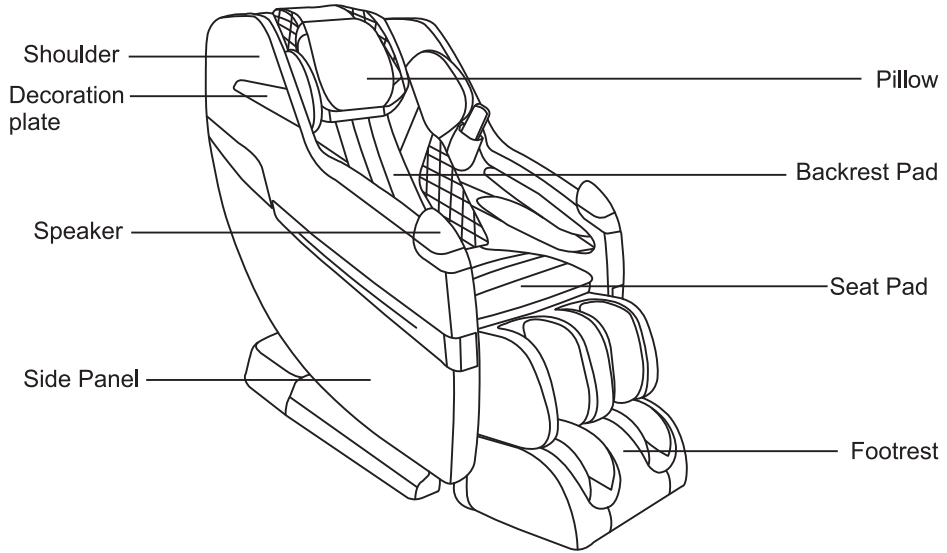
- 
27. If the power plug is damaged, it must be replaced by a special plug available from OGAWA or authorized agents.
28. As a safety precaution, Motor will stop working automatically (for protection) if apply too strong external force during use. Once appear above, switch OFF power, and switch ON again after 10 seconds. Please pay attention to this, especially for user whose weight over 100KG.
29. The appliance has a heated surface. Persons insensitive to heat must be careful when using the appliance.
30. This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved

Warranty Claims

1. Warranty is not transferable.
2. This warranty is effective only if the product is purchased from OGAWA.
3. Proof of purchase (original receipt) is required for all warranty repairs.
4. All implied warranties, including but not limited to these implied warranties of fitness and merchantability, are limited to the duration stated in the warranty policy.
5. Damage due to misuse, improper treatment and unauthorized modification and repair are not covered by this warranty.
6. Warranty does not cover accessories and attachments which do not belong to the massage unit.
7. Warranty is not effective to rental, business, commercial, institutional or other non-residential users.
8. All services covered by this warranty must be approved by OGAWA and repaired by authorized technicians only.
9. If replacement parts for defective materials are not available, OGAWA reserves the right to make substitution in lieu of repair or replacement.



Product Parts

Important-product image or content descriptions may slightly differ due to product quality improvement.

Moving the Chair

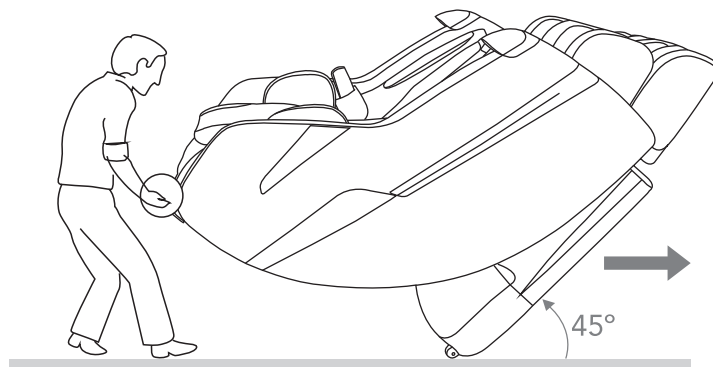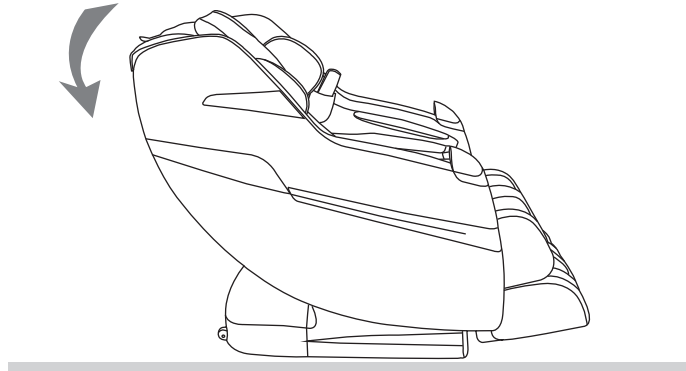
Press down the backrest, before reclining the chair to approximate 45° to easily move the chair around the house with a slight pull or push.

Caution:

- i. Before moving the chair, please put the footrest back in the original position. Otherwise, injuries might occur in the process of moving the chair.
- ii. Avoid moving the massage chair when there is someone seated on it.
- iii. Placing a mat underneath the massage chair is advisable, as the moving process might leave scratches on the floor surface.

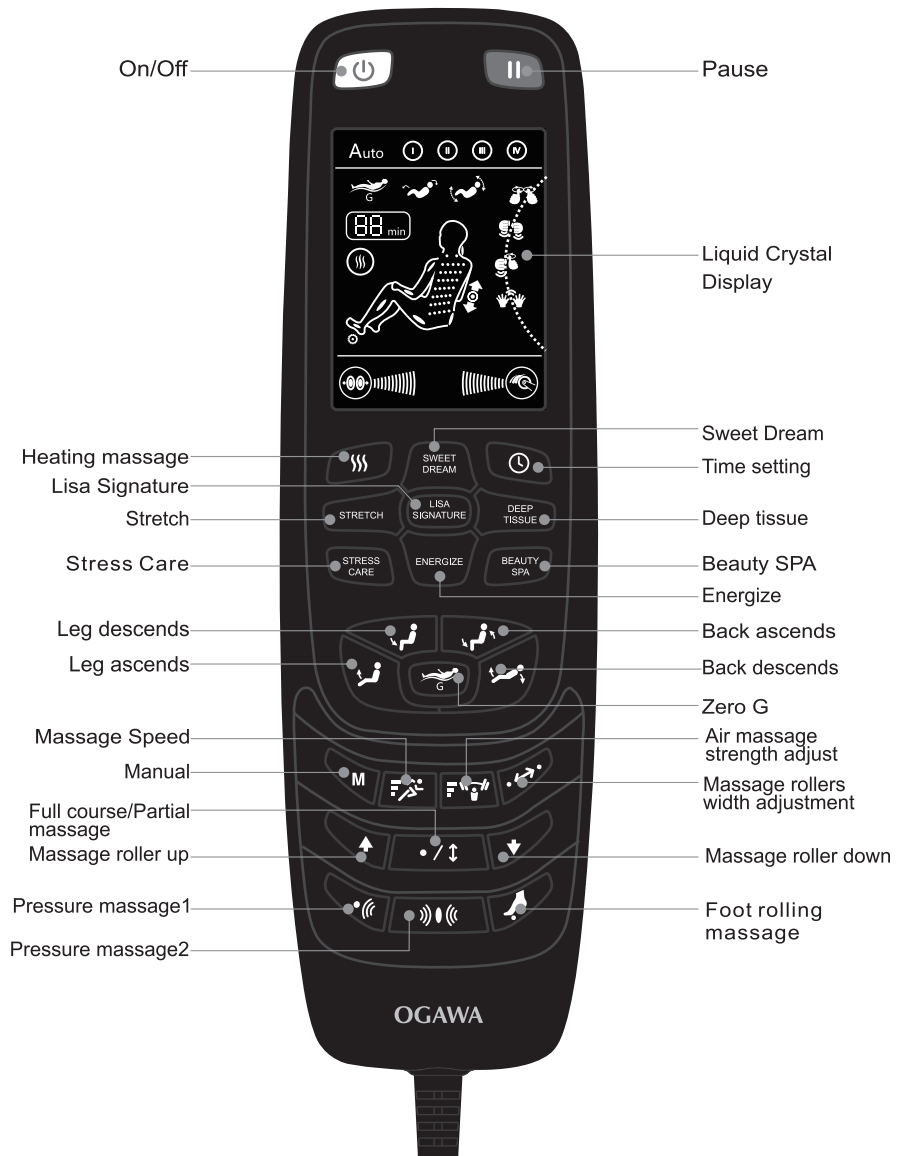
Controller Diagram

Product Operation Guide

1. Put on the power and turn to “I” the position of switch.

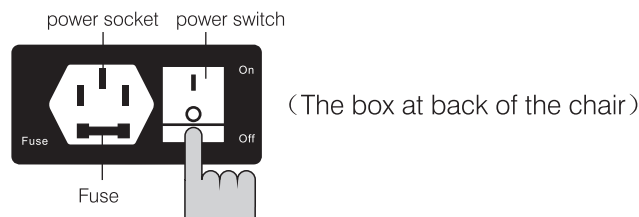Connect power line on the socket on the box at back of the chair, and then connect power plug on the electric supply. Put on the power then turn to “I” the position of switch on the box at back of the chair. Massage chair start to restore with OFF button glittering on controller. After it, massage chair get in standby mode with OFF button lighting.

2. Seat on massage chair and press ON button to start

After seating on massage chair and pressing ON button, the massage chair get in Spa massage mode with suitable position.

On/Off

Time setting

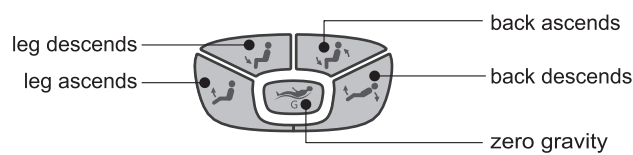
Press “Time setting” button to adjust the time by your desire. Maximum time is 30 minutes.

Unit is minute from 30 minutes to 1 minute; unit is second for countdown in last minute.

3. Adjust backrest cushion and leg rest cushion to favorable angle

When press the control keys of “back-leg ascends”, the chair start to adjust the angle of back and calf. When you adjust the angle to the appropriate position, loosen the control key, to position.

Press the “” button to start zero gravity massage, press again to stop.

4. Stop or finish massage

Normal stop : the timing function automatically fix time is 15 minutes,time out,all the program will stop and keep the setting posture.

Force stop: when press the restore during massage. it will stop massage immeditely, press again to continue.

Pause

Product Operation Guide

5. Start to massage

Automatic course

The following are the Auto Program available:

1. Sweet Dreams
2. Stretch
3. Stress Care
4. Lisa Signature
5. Energize
6. Deep tissue
7. Beauty SPA

※ For above programs, the fix time is 15 minutes and the default of airbag intensity is low.

Press “” button to adjust the intensity of air pressure.

※ The above automatic full-body massage courses can not be combined with other courses.

 **Warning:** Only rated supply 220V~ must be used to avoid damage to electronic elements.

Product Operation Guide

Manual setting course

Introduction of manual setting course

Partial air massage is the course which targets different needs and areas of body.

| Button | Description | Main function and adjustment guide |
|-----------------------------------------------------------------------------------|--------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------|
|  |  Kneading | Press one time the button to start kneading massage. The speed is adjustable. |
| |  Knocking | Press two times the button to start knead-knocking massage. The speed is adjustable. |
| |  Knead-knocking | Press three time the button to start knocking massage. The speed and width is adjustable. |
| |  Tapping | Press four times the button to start tapping massage. The speed and width is adjustable. |

※ The above manual setting course could not be combined with other courses.

※ For the above manual setting course, the default setting is spot-specific massage with medium width. Speed and width is adjustable by pressing relative button.

Function introduction for manual setting course

| Button | Description | Main function |
|-------------------------------------------------------------------------------------|-----------------------------|-------------------------------------------------------------------------------------------------------------------------------------------|
|  | Speed adjustment | There are three levels to choose from slow, medium and fast. |
|  | Width adjustment | There are three levels to choose from narrow, medium and wide. |
|  | Full course/Partial massage | Press one time the button to start full course massage . Press two times the button to start partial massage. Close by three times. |
|  | Massage roller move up | Press the button to move massage roller, loose the button to stop. |
|  | Massage roller move down | |

Product Operation Guide

Functions on buttons

Introductions on buttons used for manual setting course

| Button | Description | Main function |
|-------------------------------------------------------------------------------------|-----------------------------------|------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
|  | Air pressure intensity adjustment | Air pressure intensity is adjustable It could be adjusted only in air pressure massage. The circle of intensity is soft, medium and strong. |
|  | air massage 1 | Press one time to start air massage for full body. Press again to close the air massage. |
|  | air massage 2 | Press one time to start air massage for shoulders. Press two times to start air massage for arms. Press three times to start air massage for leg. Press four times to start air massage for foot. Close by five times. |
|  | Rolling massage | Foot rolling control key. Press the button to start foot rolling massage, press again to stop. |
|  | Heating function | Waist heating function. Press the button to start waist heating massage, press again to stop. |

Music Function

Play Mobile Music: Turn on bluetooth of mobile, then select bluetooth device of massage chair, after it automatch with massage chair speakers, it can be realized through the massage chair play mobile music function.

Product Operation Guide

End Operation

Please press the “ON/OFF” button after use, and the chair will stop its operation, while the massage rollers will return to their original position; then switch the power supply button under the chair to “O” and unplug the machine from the electrical outlet.

⚠ Warning: Supervision and guidance are necessary when disable, people who have obstacles in sense or spiritual, who are inexperienced in using it and who are lacking of the knowledge of it use the massage chair (including children). Unless there is someone who in charge of their safety, the massage chair is unavailable for them. Please take care of your children and make sure the chair not is played by them. There is heat function for the chair and should be cautions by those who insensitivity of heat.

Product Features

OGAWA massage Chair is controlled by a microcomputer that performs various intelligent massage methods and combinations, like Kneading, Swedish, Tapping, Shiatsu, Clapping, Rolling, Footroller and Air Massage. All these massage methods accurately capture the acupressure points that could alleviate fatigue, promote metabolism and improve blood circulation effectively. Equipped with eight of full body automatic program, this product thoroughly fulfills your various needs.

Various types of manual / auto massage to suit your needs.

Spring-styled massage that is specially structured for the back is designed according to the curve of the human body to provide a more effective and thorough massage.

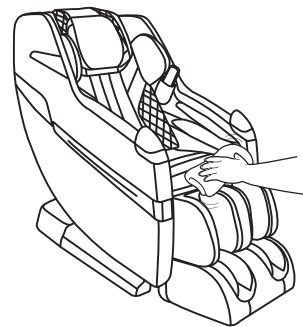
1. Designed according to body curve, to provide better massage sensations.
2. The Timer Function can be adjusted according to your desired duration of the massage programmer.
3. Liquid Crystal Display controller for a user friendly operation.
4. Built in with power surge and automatic temperature sensor for safety.
5. The operation is controlled by an intelligent microprocessor for excellent performance and consistent quality.
6. The machine radiates heat on the surface, Do Not use it over sensitive skin areas or in poorly-ventilated areas.

Cleaning And Maintenance

Cleaning of the synthetic leather

Wipe the synthetic leather with a soft dry cloth.

- **Do not clean the product with chemicals such as thinners, gasoline and alcohol.**
- **When using a leather maintenance product (cleaning cloth) sold on the market, follow its directions for use.**
- **If the synthetic leather is very dirty, wipe it with the method below.**
 1. Put a piece of soft cloth into water or 3% to 5% neutral detergent contained hot water and wring it out.
 2. Wipe surface of the synthetic leather with the soft cloth.
 3. Wipe the detergent with a piece of cloth once soaked in clean water and wrung out.
 4. Wipe with a piece of wrung soft cloth.
 5. Have the synthetic leather take natural air drying.
- **If it is difficult to remove the stains on the synthetic leather, soak a piece of melamine foam bought from the market into a neutral detergent and use it wipe the product.**
- **Do not dry the surface by a hair drier.**
- **Surface of the synthetic leather may be dyed by fabric colors; therefore, be careful when wiping the product with denim or colored cloth.**
- **Keep these positions from long-time contact with plastics to avoid fading.**
- **Synthetic leather may discolor, so the position in contact with the hair dye used should be covered by a towel.**

Cleaning of the plastic parts

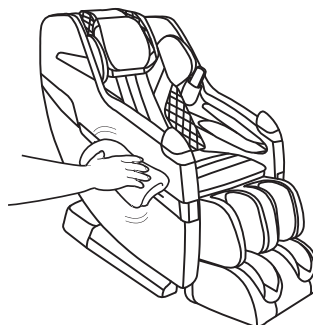
1 Put a piece of soft cloth into water or neutral detergent contained hot water, wring it out and wipe the plastic parts.

- Do not clean the product with chemicals such as thinners, gasoline and alcohol.

2 Wipe with a piece of cloth once soaked in clean water and wrung out.

- Wring the cloth out before using it to wipe the remote controller.

3 Have the plastic parts take natural air drying.

Questions and answers(Q&A)

Q Can persons paying frequent visits to hospitals because of diseases use the product?

A **These persons should consult physicians before using the product.**
 Massage is the so-called "touch-pressure stimulation": Muscles are compressed to relieve their tension and promote hemokinesis. Some diseases may get worsened because of massage. Thus, consult the attending doctors before using the product.

Q How about the body height requirement for use of the product?

A **The body height requirement differs with the specific body type. See the rough reference below when using the product.**

If a person shorter than 145 centimeters or taller than 185 centimeters uses the upper body massage function and the Lower Leg/Foot massage function of the product at the same time, it will be possible that his soles will have poor contact with the foot massage section or that his knees will need to be bent.

For a taller person with knee bending when using the product:

Select method **A**

A Tilt the backrest and lower the Lower Leg/Foot massage section.

For a short person difficult to touch the foot massage section:

Select method **A**

A Raise the backrest and lift the Lower Leg/Foot massage section.

Q Are there any things needing special attention during moving?

A **Pay attention to the following things during moving.**

For convenience during moving, tilt backrest of the product till the lowest position.

During handling, do not sit on the product.

① Press ON button power on the product.

② Press  to make the backrest completely tilt.

Press and hold on these two buttons until backrest and footrest stop 

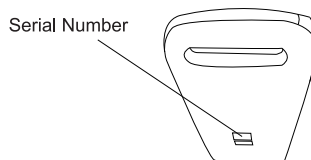
③ Turn off the power switch and pull out the power plug.

● If the original packing box is unavailable, make sure the power line is not wound on the product, wind it and place it on the seat surface. When handling the product, keep the power line from falling off and rubbing with ground.

● If the original packing box is unavailable, properly keep other packing auxiliary materials and wrap the product with them to avoid scratch and vibration during handling.

Q Where is serial number of the product?

A It is on back of the main body.

Q I care about the sounds the product makes when being used.

A It is caused by the core structure.

Q Does it matter if the massage wheels have different massage intensities?

A **This phenomenon possibly exists during massage because of the core structure.**

To simulate the most natural massage effect, the product is designed with a core structure allowing alternate back massage by a left massage wheel and a right massage wheel. During massage, the massage wheels possibly have slightly different positions on the back. That's why you have that feeling. It is not a fault.

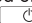
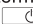
Q The acupressure force is too large.

A **Set the air bag intensity to be "1" through the Air Bag button. You can also lower the Lower Leg/Foot massage section to reduce the acupressure force on the feet.**

Troubleshooting

After a fault happens, inspect and try the corresponding method below.

If the fault still exists after that, contact the shop from which you bought the product.


| Fault | Cause and elimination method |
|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <p>The massage wheels stop during use (The product beeps and all icons and buttons on the remote controller flicker and then go out.)</p> | <p>For safety, the massage wheels will possibly stop upon a too large external force during use. If all marks and buttons on the remote controller begin to flicker, it suggests an extreme abnormality. In this case, turn off the power switch and turn it on about 10 seconds later. If the massage wheels stop again, adjust your sitting posture and do not lean on the backrest completely. (Users, particularly those heavier than 100 kilograms, should follow relevant precautions.)</p> |
| <p>The product cannot tilt, or the footrest cannot be raised or lowered (The product beeps, the running light on the remote controller goes out and the product stops)</p> | <p>For safety, the product will stop upon an obstruction or a too large force. If all contents and buttons are flickering and the backrest or the footrest has stopped, turn off the power supply and turn it on about 10 seconds later and then operate the product.</p> |
| <p>The massage wheels cannot reach the shoulders or the neck</p> | <p>If the head is not in contact with the headrest, or the back is not in contact with the backrest, the shoulder position in body type detection will possibly be lower than the actual position. Sit in the deepest position of the product, lean your head on the headrest and operate from the beginning again.</p> |
| <p>The left and right massage wheels have different heights (intensities)</p> | <p>It is not a fault but a normal phenomenon because the product is provided with a massage core allowing alternate beats of the massage wheels.</p> |
| <p>The product cannot run at all. ● Upper body (the neck to the waist) massage ● Lower leg/foot massage</p> | <ul style="list-style-type: none"> ● The power line is disconnected. ● The power switch is not turned on. ● No program is selected after  on the remote controller is pressed. |
| <p>No warm feeling in knee.</p> | <ul style="list-style-type: none"> ● The heating button is set to be OFF. ● The warm feeling depends on factors such as body type, massage method, clothes and room temperature. Make the knee in close contact, the warm transfer part is in close contact with the knee to make you feel warm. You may cover your knees with a blanket to get warmer. |
| <p>The product cannot cool down even if the heating function is disabled.</p> | <ul style="list-style-type: none"> ● Due to the heater structure, the product will still feel warm shortly after heating because of the residual heat. ● If a single part is continually massaged even after the heating function is disabled, you will feel increasingly warmer. |
| <p>The power supply cannot be connected even if  is pressed again after the massage</p> | <ul style="list-style-type: none"> ● The program will terminate about 5 seconds after the massage has ended. Press  at least 5 seconds after all the movements have stopped. |
| <p>The product is damaged</p> | |
| <p>The power line or the power plug has abnormal heating</p> | <p>To avoid accidents, contact an authorized service center.</p> |

Troubleshooting

The sounds and feelings you care about in use of the product

You will hear or have the following sounds or feelings when using the product. It is not abnormal phenomena but results of the product structure.

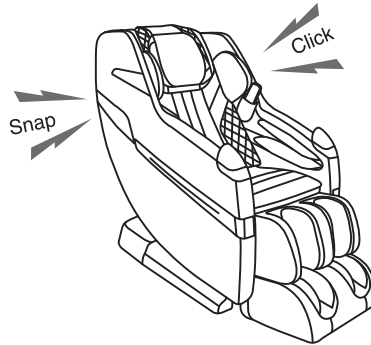- Sound when the stretching massages are retracted
- Sound and feeling when the massage wheels rub patterns on the fabric
- Sound of movements such as kneading and beat
- Friction sound of the running massage wheels
- Sound of up-down movement
- Sound of the conveyor belt
- Motor sound
- Sound of the squeezing massage wheels when they begin to stretch
- Sound when a person is sitting on the seat
- Friction sound between the armrests and the back cushion when the backrest is tilting
- Friction sound between the massage wheels and the fabric
- Sound of the inflator pump during Air Bag massage
- Sound of air exhaust during Air Bag massage
- Running sound of the solenoid valve at bottom of the product during Air Bag massage.
- Swing of the footrest
- Sound when the shoulders are stretched or moved
- Running sound of the air bags
- Sliding sound of the foot massage section
- Friction sound between the back cushion and the back cover leather when the backrest is tilting

If the sound is loud, it is also possible that the product has failed.

- In this case, have the product inspected and repaired by an authorized service center.

Warning

- **Do not refit, disassemble or repair the product without permission.** Otherwise fires, abnormal movements or personal injuries may happen.

Warning

- **In the case of any abnormality or fault, stop the product immediately and pull out the power plug to avoid fuming, fires and electric shocks.**
< Abnormality and fault examples >
 - The product fails to work after the power switch is turned to the ON position;
 - After the power line is moved, the product is discontinuously powered on;
 - In use, the product gives off a burning smell or an abnormal sound;
 - The product is deformed or abnormally heating→ In any of these abnormalities and faults, have the product inspected and repaired by an authorized service center immediately.

Product Specifications

| | | |
|---------------------------------------|--------------------------------------|--------------------------------------------------------------------------|
| Power supply | | AC220-240V~,50/60Hz |
| Power consumption | | 150W |
| Upper body massage | Range of massage (up-down direction) | Approx. 97cm |
| | Massage speed | Kneading:Approx.21 times/min-Approx.37 times/min |
| | | Beat:Approx.422 times/min-Approx. 703 times/min |
| | | Moving speed:Approx.43mm/s |
| Power consumption | | Backrest: Approx. 124°~157° Footrest: Approx. 92.5°~ 115° |
| Automatic power-off | | Approx. 15 minutes(automatic power-off) |
| Dimension | When it is not tilting (H x W x L) | Approx. 1520mm x 785mm x 1175mm |
| | When it is tilting (H x W x L) | Approx. 1935mm x 785mm x 1025 mm |
| Weight | | Approx. 90 Kg |
| Dimensions of packing box (H x W x L) | | Seat rest: 1420x820x1235mm Footrest: 530x440x510mm |
| Weight(including the packing box) | | Approx. 106 Kg |
| Operating Conditions | | Ambient temperature:Approx.0°C~35°C Relative humidity:Approx.20%~80% |
| Storage Conditions | | Ambient temperature:Approx.-5°C~35°C Relative humidity:Approx.20%~80% |
| Maximum body weight of user | | Approx. 120Kg |

For a user heavier than 100 kilograms, the product may have bigger noise and its fabric housing may be worn quicker.

Contact Us

MALAYSIA
Healthy World Lifestyle Sdn Bhd
No. 22 Jalan Anggerik Mokara 31/47
Kota Kemuning, 40460 Shah Alam
Selangor Darul Ehsan, Malaysia
Tel : +603-5121 4286
Fax: +603-5121 4386

HONG KONG, CHINA
OGAWA Health Care International (HK) Ltd
31/F,YHC Tower,1Sheung Yuet Road
Kowloon Bay
Kowloon
Hong Kong
Tel:+852 3465 6262
Fax:+852 2410 0119

SINGAPORE
OGAWA Health-Care Pte Ltd
27 Tampines Industrial Avenue 5,
T5 @ Tampines
Singapore 528623

Tel:+065-6846 8588
Fax:+065-6846 7677

VIETNAM
OGAWA Vietnam Co Ltd
2nd floor,Petroland Tower,
12 Tan Trao,Tan Phu ward
District 7, Ho Chi Minh
Tel:+84 8 54133 222/54133 223
Fax:+84 8 5416 1736

PHILIPPINES
O-Healthcare Solution Phil. Inc.
2303, 4th Floor, G&A Building,
Chino Roces Avenue Extension,
Makati City, Philippines
Tel : +632 889 0676 / + 632 844 6851
Fax: +632 812 0421

SMARTJAZZ

Energized with melody touch 爵士椅

User Manual
All information is correct at the time of print

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Korea . Australia . Dubai . K.S.A . Nigeria . Egypt . Lebanon . Denmark . Sweden . U.S.A